
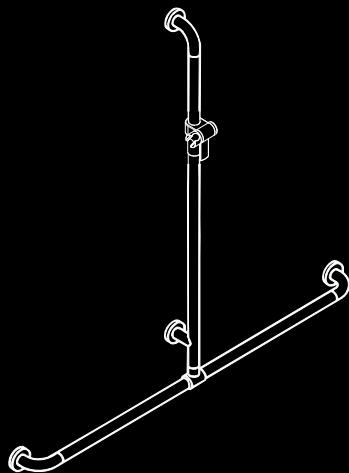
- Ⓓ **Montageanleitung**
- ⒼⒷ **Installation instructions**
- Ⓕ **Montage**
- ⓃⓁ **Montage van**
- ⓖ **Istruzioni di montaggio**
- Ⓔ **Montaje instrucciones**
- Ⓟ **Instrukcja montażu**


Artikel-Nr.
item number
No de référence
Type Nr.
Nr. Articolo
N° de artículo
Numer produktu

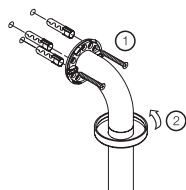4 - 5

Montageanleitungen
Installation instructions
Notice de montage
Montage van
Istruzioni di montaggio
Instrucciones de montaje
Instrukcje montażu

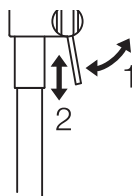6 - 9

Montagehinweis
Mounting information
Conseil de montage
Montage aanwijzing
Indicazioni per il montaggio
Indicaciones de montaje
Wskazówka dotycząca montażu

10 - 19

135

Zweckbestimmung
proper use
affectation
toepassingen
conformità di norma
Uso previsto
Przeznaczenie

12 - 13

Warnhinweise
Warning
Mises en garde
Let op
Avvertenze importanti
Advertencias
Ostrzeżenia

14 - 17

Pflegehinweise
Care tips
Conseils d'entretien
Onderhoudsinstructies
Consigli per la pulizia
Indicaciones para el mantenimiento
Pielęgnacja

18 - 26

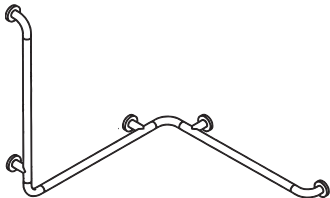
805.35.100


805.35.110
805.35.120


805.35.200

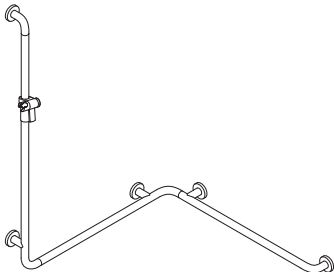
805.35.210

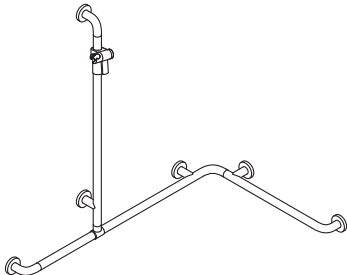
805.35.300
805.35.310

CE 805.22.100L
805.22.100R

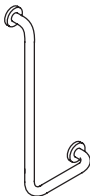
CE 805.22.110L
805.22.110R

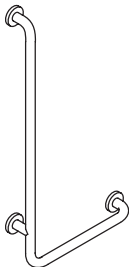
805.22.1S

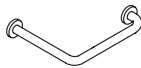
805.22.200L
805.22.200R

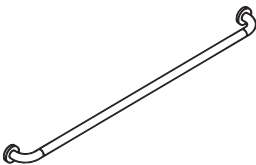805.36.100
805.36.110
805.36.120
805.36.130
805.36.140
805.36.150

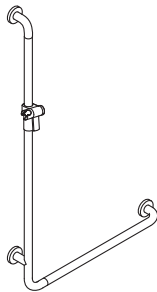
805.33.210


805.33.100
805.33.110
805.33.120

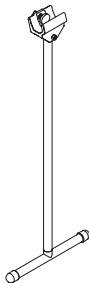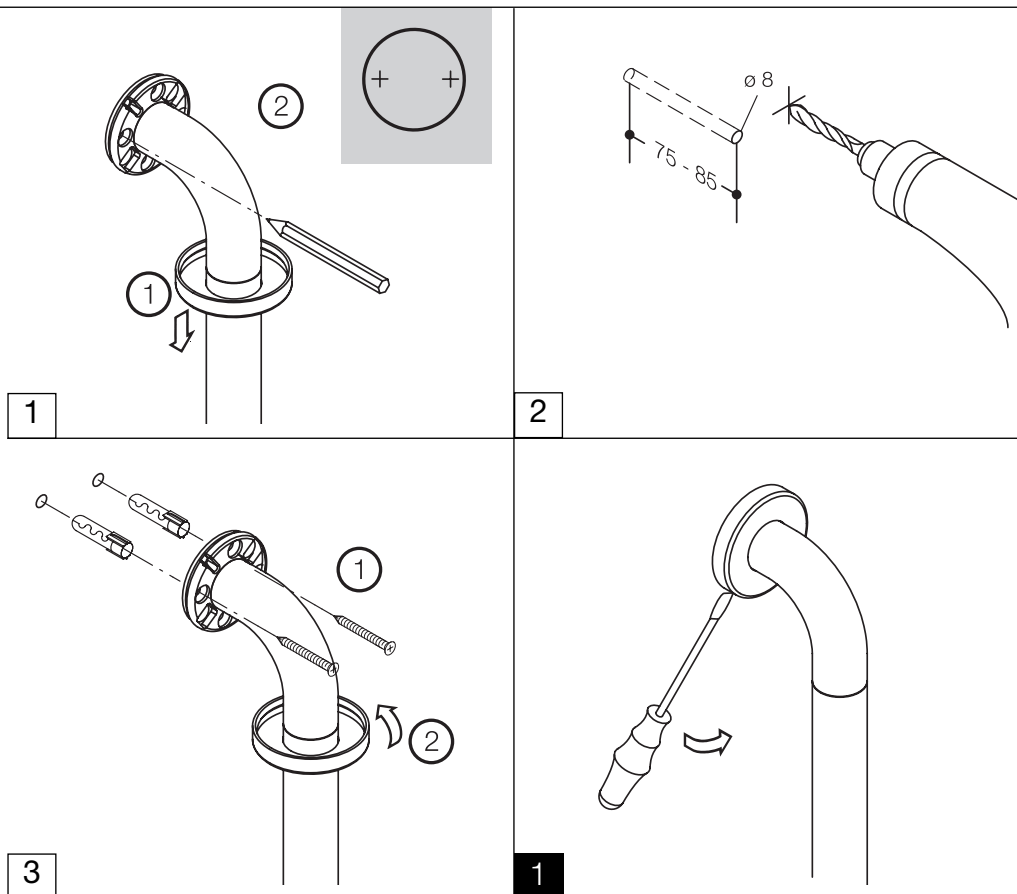802.99.100

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(805.22.100, 805.22.110, 805.22.200).

(GB)

Proper use in accordance with Directive 93/42/EWG

HEWI L-shaped support rails are designed to be used in bathrooms and toilets.

They are particularly useful for patients in hospitals or persons with a physical disability who can hold on to them when they get into a bathtub or sit down on a toilet or on some other seat.

They act as a support particularly for patients in hospitals or persons with a physical disability who can hold on to them when they get out of a bathtub or get up from the toilet or from some other seat. The toilet roll holder is an optional extra that can be fitted to the support rail to accommodate a toilet roll and thus make it more easily accessible particularly for hospital patients or persons with a physical disability.

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Warning

Use of HEWI shower and bath grab rail

The HEWI shower and bath grab rail is designed to withstand a maximum static working load of 150 kg. Do not use this support rail for weights in excess of this. Any non-compliance with loading limits can cause permanent damage to the support rail. In such cases, the support rail must be examined by an specialised company prior to the support rail being used again. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

Only use the HEWI shower and bath grab rail in the manner described in the instructions. Any improper use e.g. as a gymnastic apparatus or climbing aid can cause an accident with persons potentially suffering serious injury. Assuming normal use, the support rail is of a flexible nature thanks to the materials used in its manufacture. This flexibility reduces the risk of injury if anyone bumps into the support rail. The fact that the support rail bends to a certain extent under load is normal. Should however the support rail become permanently deformed, you should immediately contact the company appointed to carry out maintenance work or HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300.

Installation of HEWI shower and bath grab rail

The smooth and safe functioning of the HEWI shower and bath grab rail largely depends on its correct fixing to the given wall, which is why this work should only be entrusted to a licensed contractor (sanitary fitter). Serious impairment to the support rail's proper functioning can occur if its installation is not carried out professionally, potentially resulting in accidents with persons suffering serious injury. This is why we advise you not to install the support rail yourself.

When the licensed contractor mounts the HEWI shower and bath grab rail, he should ensure that the entire area of the wall to which the wall mounting plate is to be fitted is flat (no projecting joints or tiles sticking out), that the structure of the wall is suitable for the installation of the support rail and has no weak points. Furthermore, the choice of the right fixing material kit for the job is important; HEWI offers various types to suit the nature of the wall concerned. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

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GB

Care tips

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough. If, however, you feel you have to use detergents, you should take note of the following information. Only use detergents with a pH-rating of between 6 and 8.

Detergents containing active substances such as phosphates, soap and tensides may be used. Detergents containing acids, alkalis, bleach or scouring agents may not be used. Nor should utensils with a scouring effect such as brushes etc. be used.

Usage of any detergent requires full compliance with the manufacturer's instructions (concentration, soaking time etc.).

If this is not done, the surface finishes of the HEWI products may suffer as a result and sustain damage. HEWI accepts no liability whatsoever for damage occurring from incorrect handling

For further information please contact HEWI Technical Support, tel. +49 5691 82-300.

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Cleaning and maintenance instructions for rust-proof steel

Maintenance and cleaning of the surface ¹⁾

Maintenance and cleaning means the cleaning of finished and utilised items made of rustproof steel for optical or hygienic as well as value maintenance reasons. As a rule, one can safely say that the hard, tight and non-porous surface of rustproof steel is insensitive and easy to clean. The material is durable in its entire cross section and does not require any protective coats that could peel off or be chipped off. The excellent corrosion resistance also allows for cleaning with highly-active cleaning agents required for disinfection and sterilisation.

Slight contaminations ²⁾

Cleaning and maintenance in a household start with the removal of finger prints, dust, water stains or food leftovers. Here in most instances water and soap or cleaning agents used in private households are sufficient; soft brushes ¹⁾ or cloths are used for cleaning purposes. Depending on the water hardness, stains will remain on the steel surface when water drops dry up. If these stains spoil the image, the affected item should be wiped with wash leather (dried) after washing-up or sprinkling with water. There are no objections to the application of metal preservatives containing conditioners (e.g. hard polishing wax) as well as active cleaning components; they should be applied to the surface with a cloth extensively and in a very thin film.

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HEWI Heinrich Wilke GmbH