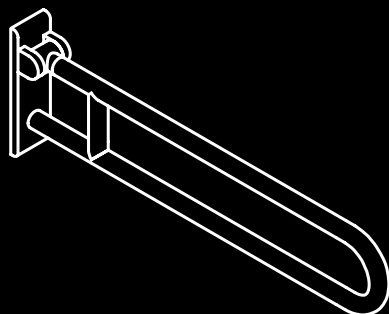
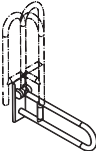
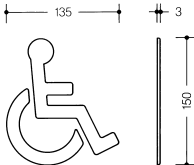
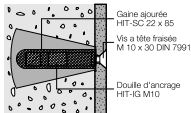
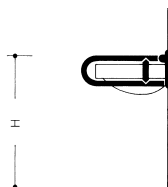
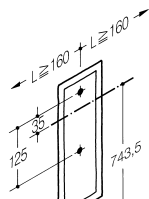
- Ⓓ **Montageanleitung**
- ⒼⒷ **Installation instructions**
- Ⓕ **Montage**
- ⓃⓁ **Montage van**
- Ⓢ **Istruzioni di montaggio**
- Ⓔ **Montaje instrucciones**
- Ⓟ **Instrukcja montażu**

	Artikel-Nr. item number No de référence Type Nr. Nr. Articolo N° de artículo Numer produktu	4
	Warnhinweise Warning Mises en garde Let op Avvertenze importanti Advertencias Ostrzeżenia	6 - 9
	Zweckbestimmung Proper use Affectation Toepassingen Conformità di norma Usò previsto Przeznaczenie	10 - 11
	Befestigungsarten Fixing types Types de fixations Bevestigingswijzen Sistemi di fissaggio Tipos de fijación Rodzaje mocowań	12 - 20

Montagehinweis
Mounting information
Conseil de montage
Montage aanwijzing
Indicazioni per il montaggio
Indicaciones de montaje
Wskazówka dotycząca montażu

22 - 32

Montageanleitungen
Installation instructions
Notice de montage
Montage van
Istruzioni di montaggio
Instrucciones de montaje
Instrukcje montażu

33 - 43

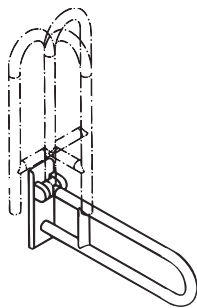
Wartung
Maintenance
Entretien
Onderhoud
Manutenzione
Mantenimiento
Konserwacja

44 - 47

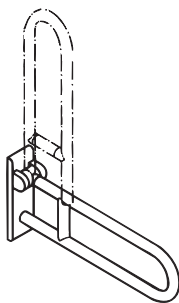
Pflegehinweise
Care tips
Conseils d'entretien
Onderhoudsinstructies
Consigli per la pulizia
Indicaciones para el mantenimiento
Pielęgnacja

48 - 53

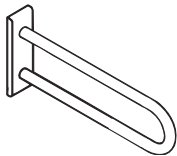
805.50.100
805.50.110
805.50.120
805.50.130

805.50.105
805.50.115
805.50.125

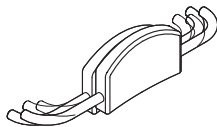
805.50.100
805.50.110
805.50.120
805.50.130

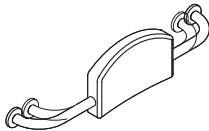
805.50.105
805.50.115
805.50.125

805.50.305
805.50.315
805.50.325
805.50.335

805.51.900

805.51.910

Warnhinweise!

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WARNING!

Use of HEWI Hinged support rail

The HEWI hinged support rail is designed to withstand a maximum static working load of 100 kg (vertical) and 35 kg (horizontal), the equivalent of 150 kg body weight assuming normal use. Do not use this support for weights in excess of this. Any non-compliance with loading limits can cause permanent damage to the support or the joint. In such cases, the support must be examined by the company appointed to carry out maintenance work prior to the support being used again. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

Only use the HEWI hinged support rail in the manner described in the instructions. Any improper use e.g. as a gymnastic apparatus or climbing aid can cause an accident with persons potentially suffering serious injury.

No persons (particularly children) may be in the area underneath the support when it is being folded down as this can cause an accident with serious injury

The fact that the support bends to a certain extent under load is normal. Should however the support become permanently deformed, you should immediately contact the company appointed to carry out maintenance work or HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300.

Installation of HEWI Hinged support rail

The smooth and safe functioning of the HEWI hinged support rail largely depends on its correct fixing to the given wall, which is why this work should only be entrusted to a licensed contractor (sanitary fitter). Serious impairment to the support's proper functioning can occur if its installation is not carried out professionally, potentially resulting in accidents with persons suffering serious injury. This is why we advise you not to install the support yourself.

When the licensed contractor mounts the HEWI hinged support rail, he should ensure that the entire area of the wall to which the wall mounting plate is to be fitted is flat (no projecting joints or tiles sticking out), that the structure of the wall is suitable for the installation of the support and has no weak points. Furthermore, the choice of the right fixing material kit for the job is important; HEWI offers various types to suit the nature of the wall concerned. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance. The instructions accompanying the wall plugs should be observed in full.

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**Proper use in accordance
with Directive 93/42/EWG**

Aids for barrierfree usage

Stationary supports are generally specified for bathrooms where they tend to be located on both sides of washbasins and toilets. They can however be used elsewhere if the need arises.

They provide users with support and something to hold on to, thus preventing them from accidentally falling off the toilet. Furthermore, persons with a physical disability can use them to help themselves stand up, enabling them to make use of bathroom facilities without help.

The toilet paper holder integrated into the front part of the support, depending on the type, enables the user to remove toilet paper with one hand and can be mounted in a left hand or right hand position or, if required, on both sides of the toilet.

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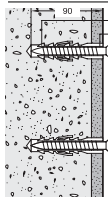
It is absolutely necessary to consider the appropriate assembly instructions and further referrals on page 12 -13.

Serie 805

Suitable for fixing to walls of:

- Beton (>B15)

Stationary support rail
Hinged support rail
Shower spray guard

BM10.2

Supplied with:
2 bolts,
2 wall plugs

Hinged seat

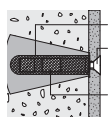
BM20.4

Supplied with:
4 bolts
4 wall plugs

Back rest

BM10.2.1

Supplied with:
2 bolts
2 centering sleeves
2 wall plugs

- solid bricks of
lightweight
concrete
- aircrete
- sand-lime
perforated bricks

BM11.2

Supplied with:
2 bolts
2 basket sleeve
2 anchor sleeve

BM21.4

Supplied with:
4 bolts
4 basket sleeve
4 anchor sleeve

BM11.2.1

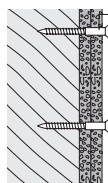
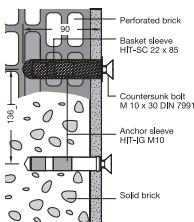
Supplied with:
2 bolts
2 centering sleeves

With basket sleeve

- vertically-perforated
bricks
- hollow blocks
- sand-lime
perforated bricks
- other
perforated bricks

Without basket sleeve

- concrete
- solid bricks
- solid pumice and
other solid materials
- natural stone
- sand-lime
solid bricks

Suitable for fixings to light-weight partition walls with support frame of hardwood min. 20 mm thick.

BM12.2

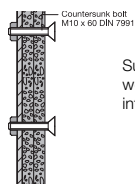
Supplied with:
2 bolts

BM22.4

Supplied with:
4 bolts,

BM12.2.1

Supplied with:
2 stainless steel locknut
2 centering sleeves

Suitable for fixings to light-weight partition walls with integrated steel plates.

BM13.2

Supplied with:
2 bolts

BM23.4

Supplied with:
4 bolts

BM13.2.1

Supplied with:
2 bolts
2 centering sleeves

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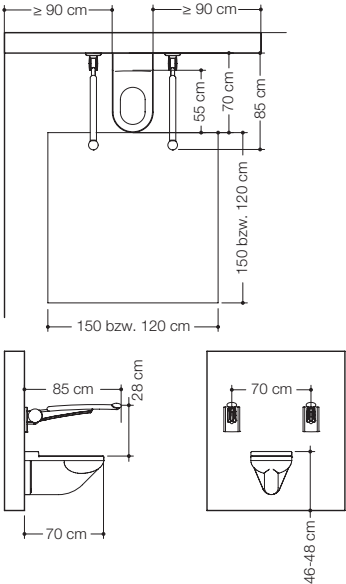
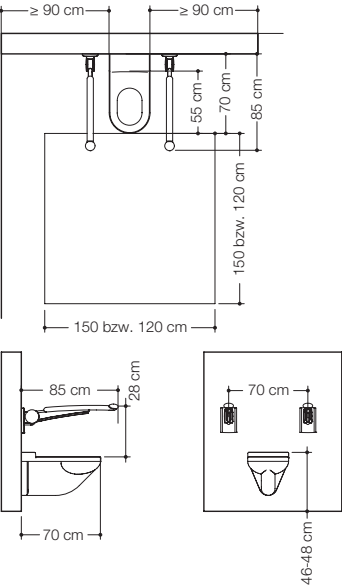
Toilet area

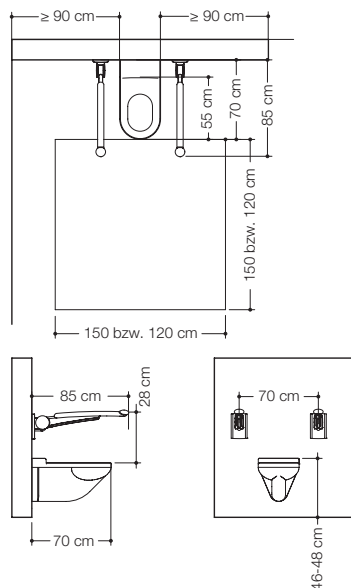
	private DIN 18 025 part 1	private DIN 18 025 part 2	public DIN 18 040 part 1
Fixing height including seat	48 cm	individual	46 - 48 cm
WC-depth (distance between lower edge and wall)	70 cm	individual	70 cm
Sideway area of movement	95 cm left or right	individual	≥ 90 cm left and right
Area of movement in front of the WC	150 cm deep 150 cm wide	120 cm deep 120 cm wide	150 cm deep 150 cm wide
Sideway distance from the WC to the wall	30 cm	individual	≥ 90 cm
Supports with lift-up feature	individual	individual	both sides, height to rim 28 cm over seat height
Fixing distance with integrated flush	individual	individual	70 cm
with integrated toilet roll holder	individual	individual	left and right left and right
Backrest rail	individual	individual	55 cm behind the front edge

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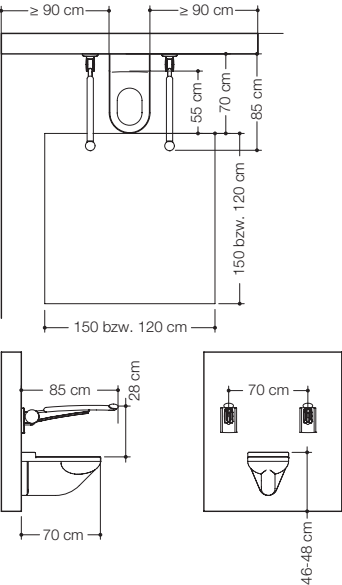
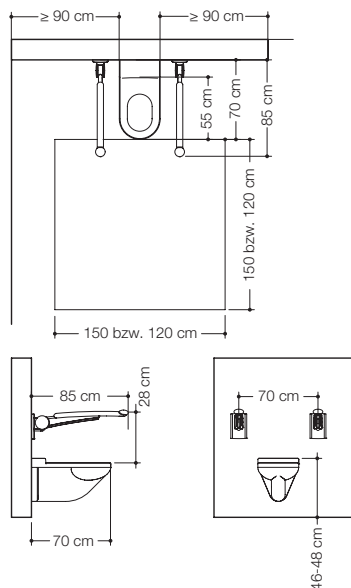
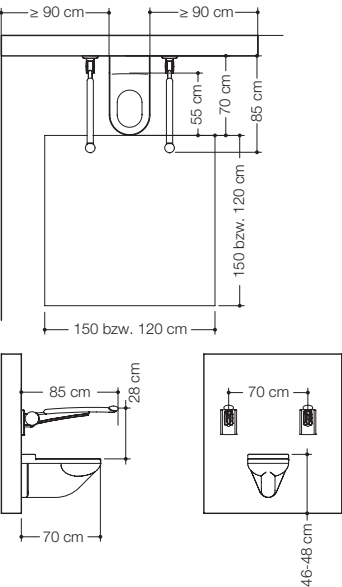
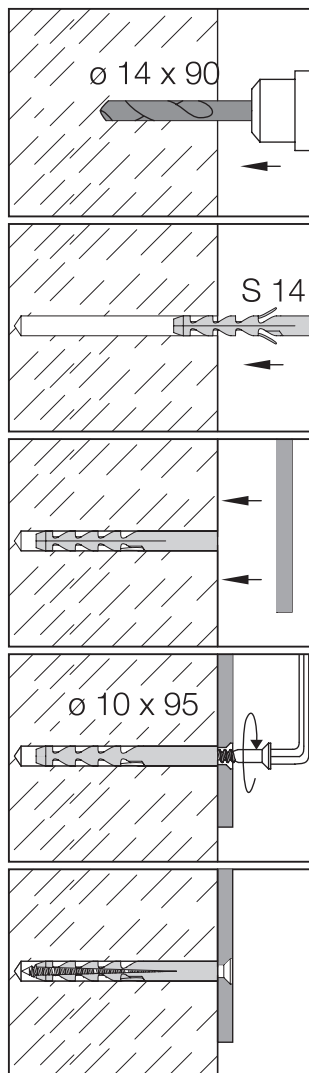
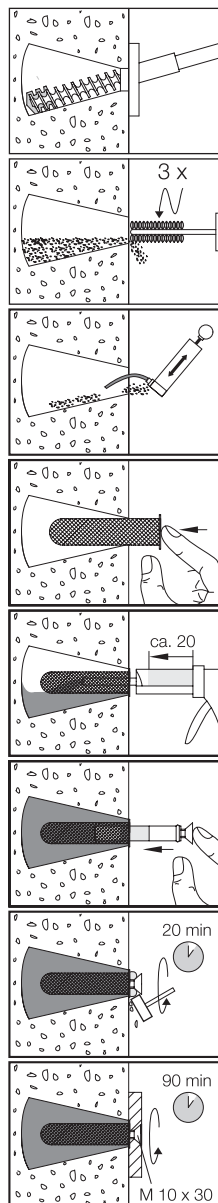
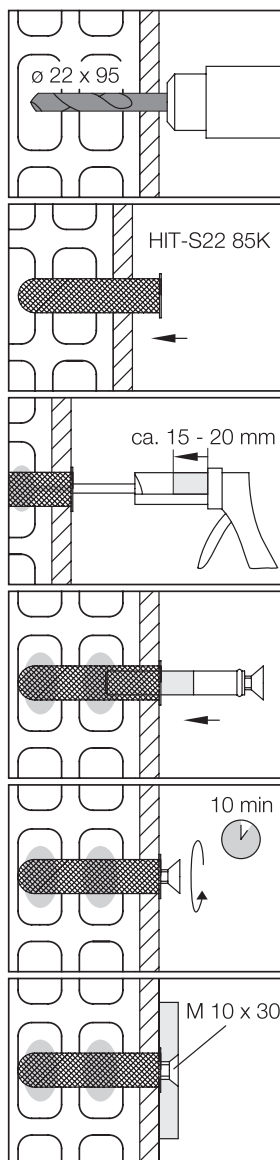
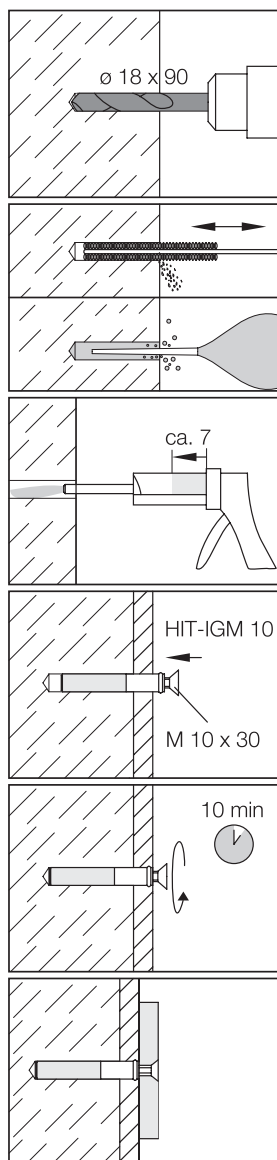
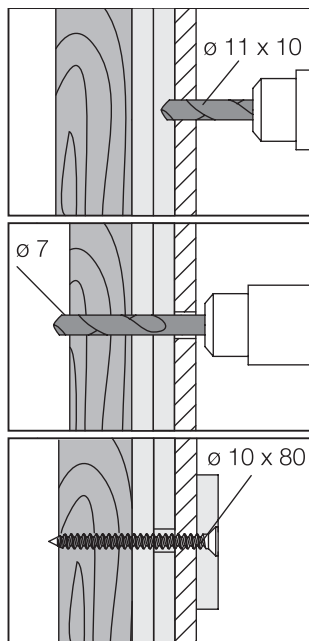
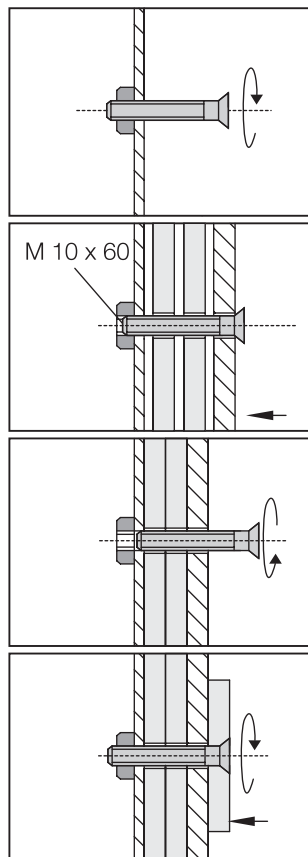
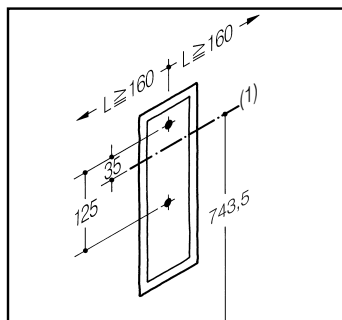
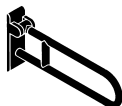
(GB) Position template and stick to the wall. Take into consideration the necessary free space $L = 160$ mm, min. between centre line and wall.
 a = Installation height between finished floor and centre of upper support rail (1).

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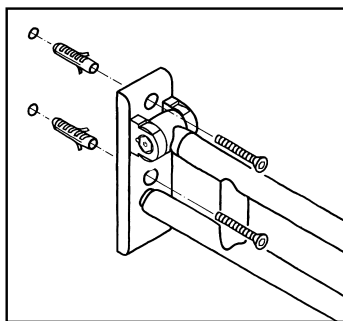
Fixing type BM10.2

Test structure of wall! Drill holes 14 mm ø and remove template. Drive in plastic plugs and tighten support.

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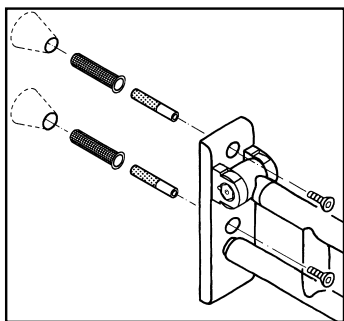
Fixing type BM11.2

Test structure of wall! Drill holes for plugs 22 mm ø, remove template. Install injection anchor and allow cement to set as per instruction (injection mortar not supplied from HEWI).

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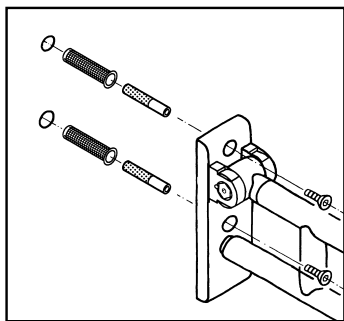
Remove protective sleeves and tighten support (protect screws against corrosion with grease).

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Fixing type BM11.2

Test structure of wall! Perforated brick:
Drill plug holes 22 mm $\varnothing$ x 95 mm
deep.

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Remove drill template. Insert filter slee-
ve and inject HIT-HY70 injection mortar
from cartridge (from local supplier; obser-
vation of manufacturer's instructions
vital). Insert anchor sleeve.

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Important! Protect M 10 internal
thread. Observe manufacturer's
recommended setting time and
tighten support (protect screws
against corrosion with grease).
Snap on caps.

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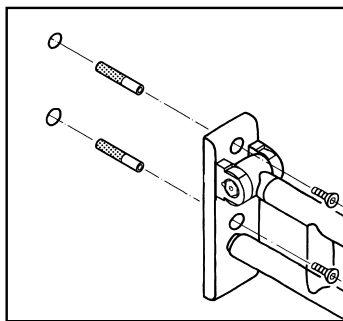
Fixing type BM11.2

Test structure of wall! Solid brick: Drill plug holes 18 mm $\varnothing$ x 90 mm deep. Remove drill template. Ensure plug holes are clean and free of dust.

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Apply injection mortar or from cartridge (from local supplier; observation of manufacturer's instructions vital). Insert anchor sleeve.

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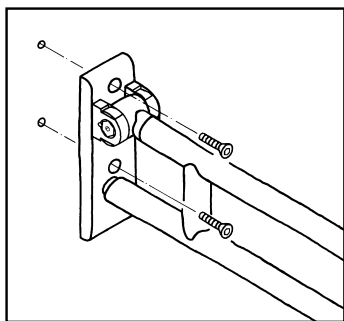
Important! Protect M 10 internal thread. Observe manufacturer's recommended setting time and tighten support (protect screws against corrosion with grease).

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Fixing type BM12.2

Test structure of wall! Drill holes for screws 7 mm ø (if mounted on tiles, drill 11 mm ø clearance hole through tiles). Tighten support.

Tighten support. Snap in caps. Insert anchor sleeve.

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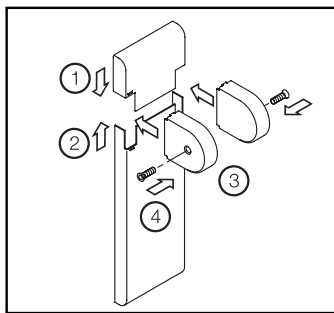
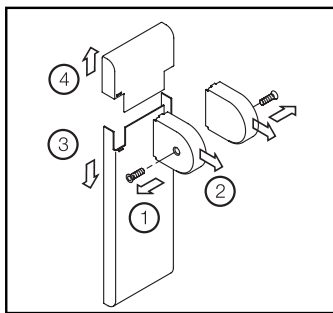
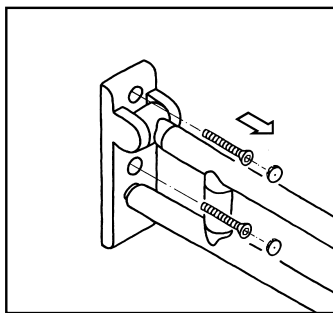
Abdeckung demontieren.

Demontage

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Cover for support rails with lift-up feature

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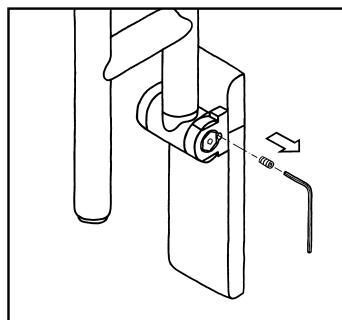
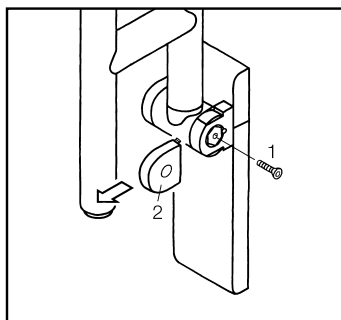
Brake adjustment for hinged
 support rail

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(GB) Direction of travel downwards
 Unscrew screw (1). Pull off right-hand cap (2).

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Turn out the M 5 setscrew with a size 2.5 Allen key.

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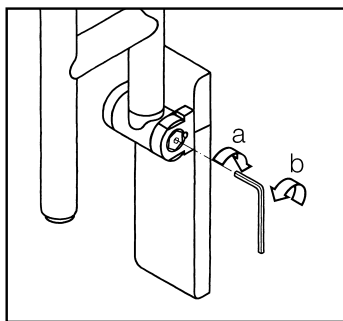
Turn the axial screw with a size 4 to direction a to strengthen the braking force, to direction b to decrease the braking force.

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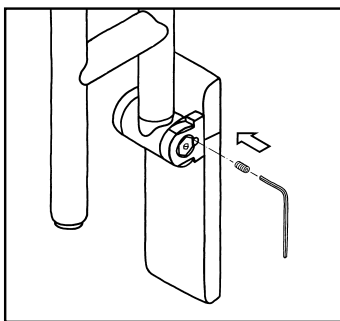
Set the M 5 setscrew back into position. **Warning:** the setscrew must be seated flush.

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Maintenance of hinged support rail

The HEWI hinged support rail is not maintenance free! The correct functioning of the brake device that prevents the support from dropping accidentally can be impaired by external influences (temperature, moisture). The support ought to be inspected by the maintenance company at least once a month to ensure its correct functioning. The persons entrusted with the maintenance work must be familiar with the functionality of the support and the correct setting of the brake device. The appropriate instructions and description are enclosed with every support. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance. Do not carry out the maintenance work yourself as this could lead to the correct functioning of the support being severely impaired with the risk of persons potentially suffering severe injuries.

The maintenance work must include examination of the brake device to ensure the support folds down slowly and smoothly. If necessary, the friction device should be re-adjusted. Observe these instructions. Under no circumstances should the brake discs or joint be greased or lubricated. Moreover, the wall fixing must be inspected to ensure everything is as it should be. Non-compliance with these instructions can cause an accident with persons potentially suffering serious injury.

Damage in transit

The support left the factory in a perfect condition. When the customer takes possession of it, he/she should examine it for any damage that may have occurred in transit. Any defects have to be reported immediately to ensure that no potentially severe impairment to the support functionality can subsequently occur, thus eliminating the risk of persons suffering serious injury. The surface of the support is sensitive and can suffer scratching caused by dust or dirt on or near the installation site. Keep the support in its packaging whenever possible. Once the support has been installed, put the protective foil back around it until the support needs to be used.

Exclusion of liability

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of the improper installation, use or maintenance of the company's hinged support rail, particularly if the safety instructions and other points to note listed above are not or have not been observed in full.

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GB

Care tips

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough.

If, however, you feel you have to use detergents, you should take note of the following information.

Only use detergents with a pH-rating of between 6 and 8.

If this is not done, the surface finishes of the HEWI products may suffer as a result and sustain damage. HEWI accepts no liability whatsoever for damage occurring from incorrect handling

For further information please contact HEWI Technical Support, tel. +49 5691 82-300.

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Cleaning and maintenance instructions for rust-proof steel

Maintenance and cleaning of the surface ¹⁾

Maintenance and cleaning means the cleaning of finished and utilised items made of rustproof steel for optical or hygienic as well as value maintenance reasons. As a rule, one can safely say that the hard, tight and non-porous surface of rustproof steel is insensitive and easy to clean. The material is durable in its entire cross section and does not require any protective coats that could peel off or be chipped off. The excellent corrosion resistance also allows for cleaning with highly-active cleaning agents required for disinfection and sterilisation.

Slight contaminations ²⁾

Cleaning and maintenance in a household start with the removal of finger prints, dust, water stains or food leftovers. Here in most instances water and soap or cleaning agents used in private households are sufficient; soft brushes ¹⁾ or cloths are used for cleaning purposes. Depending on the water hardness, stains will remain on the steel surface when water drops dry up. If these stains spoil the image, the affected item should be wiped with wash leather (dried) after washing-up or sprinkling with water. There are no objections to the application of metal preservatives containing conditioners (e.g. hard polishing wax) as well as active cleaning components; they should be applied to the surface with a cloth extensively and in a very thin film.

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HEWI Heinrich Wilke GmbH