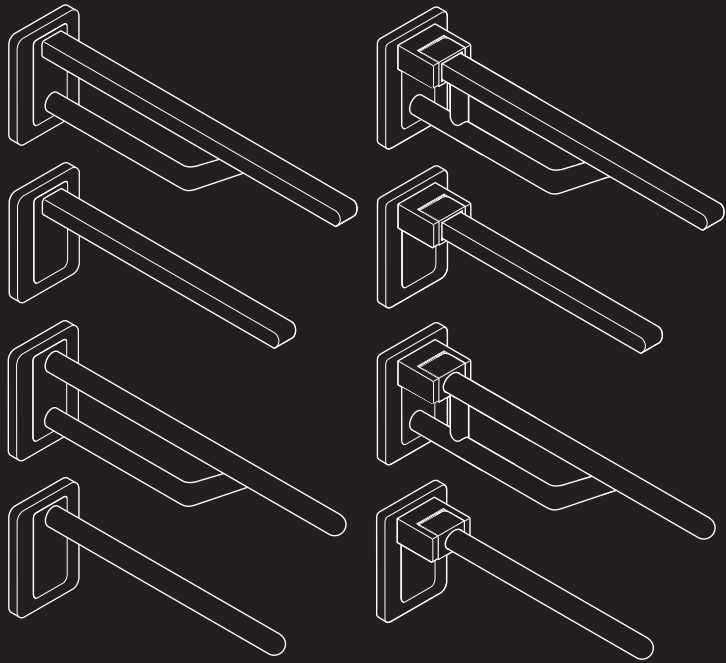
- Ⓓ **Montageanleitung**
- ⒼⒷ **Mounting instructions**
- Ⓕ **Notice de montage**
- ⒶⓁ **Montage-instructie**
- Ⓘ **Istruzioni di montaggio**
- Ⓔ **Instrucciones de montaje**
- Ⓟ **Instrukcja montażu**

(D) Inhalt

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Purpose according to Directive 93/42/EEC

The HEWI support rail is designed for use in bathrooms and lavatories. It is used as an aid to sitting down and standing up in lavatories and also offers people with physical disabilities additional support and something to hold on to.

Mounting only by a specialist firm

Mounting should be performed solely by qualified specialists (sanitary installers).

The following points must be observed:

- The area to which the support rail is to be fastened must be fully load-bearing, resilient and flat (no protruding joints or uneven tiles).
- Only mounting materials to the manufacturer's specifications may be used, which are suitable for the wall structure and purpose. Contact HEWI Technical Support in case of doubt.

Warning and hazard notices

- The HEWI support rail is designed for a maximum static load of 100 kg vertically and 35 kg horizontally.
- Improper mounting or use can lead to accidents involving serious injuries.
- A certain amount of vertical and horizontal bending is normal when the support rail is under load. In the event of damage (e.g. permanent bending), the support rail may only continue to be used following inspection by a specialist company responsible for maintenance.
- Do not misuse the support rail, e.g. as gymnastics apparatus or climbing aid.
- When folding down the support rail, ensure that nobody is located beneath the rail.

Care and cleaning instructions

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough. When using detergents, please observe the following:

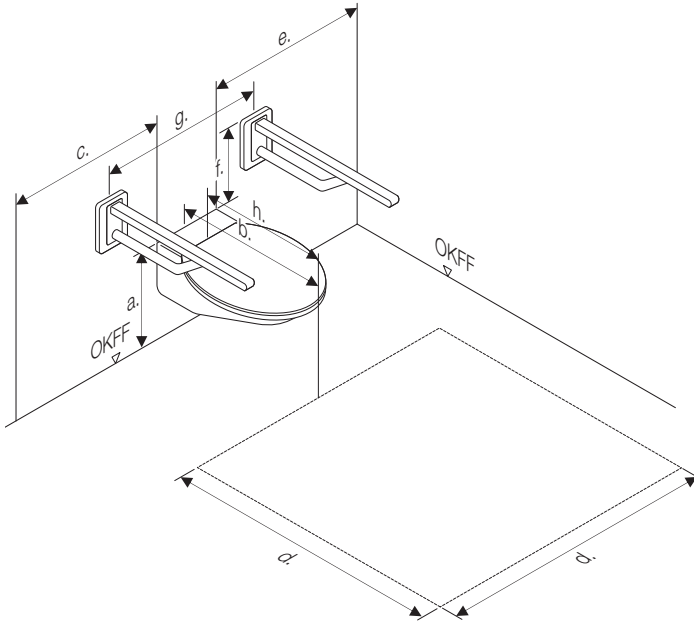
- Only use detergents with a pH of between 6 and 8.
- Detergents containing active substances such as phosphates, soap and tensides may be used.
- Follow the detergent manufacturer's instructions (concentration, contact time).
- Detergents containing acids, alkalis, bleach or scouring agents may not be used.
- Do not use scouring cleaning equipment, such as brushes

Exclusion of liability

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of improper mounting or maintenance, particularly if the safety instructions and other points to note listed are not or have not been observed in full.

HEWI accepts no liability whatsoever for damage occurring from incorrect cleaning.

**For further information, please contact HEWI Technical Support,
tel. +44 1634 377688.**

**public
DIN 18 040
part 1**

a. Fixing height including seat	46 - 48 cm
b. WC-depth (distance between lower edge and wall)	70 cm
c. Sideway area of movement	≥ 90 cm left and right
d. Area of movement in front of the WC	150 cm deep 150 cm wide
e. Sideway distance from the WC to the wall	≥ 90 cm
f. Supports with lift-up feature	both sides, height to rim 28 cm over seat height 85 cm high
g. Fixing distance – with integrated flush – with integrated toilet roll holder	70 cm left and right left and right
h. Backrest rail	55 cm behind the front edge

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(GB) Fixing type BM10.4

Test structure of wall! Drill 14 mm
rawlplug holes. Tap in rawlplugs and
screw the product firmly in place.

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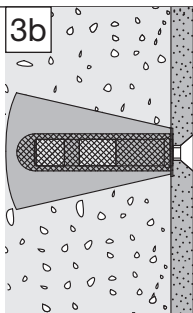
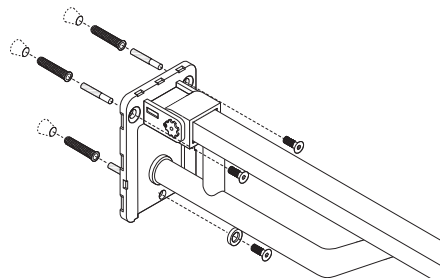
Ⓔ Fixing type BM11.4

Test structure of wall! Drill 22 mm holes with countersink and remove drilling template. Mount the rawlplugs and allow to cure in accordance with the manufacturer's instructions (put injection mortar on job site). Remove protective sheaths and screw firmly to wall (protect screws with anti-corrosion grease).

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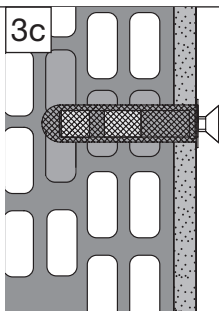
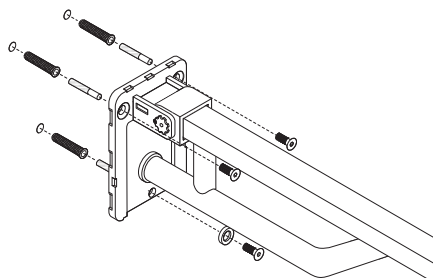
Ⓔ Fixing type BM11.4

Test structure of wall! Perforated brick: Drill 22 mm x 95 mm deep raw/plug holes. Remove drilling template. Insert perforated sleeve and inject HIT-HY70 injection mortar from cartridge (put on site – manufacturer's instructions must be followed). Insert anchor sleeve. **Warning!** Protect M 10 inner thread. Observe curing time, screw firmly to wall. Protect screws with anti-corrosion grease.

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GB Fixing type BM11.4

Test structure of wall! Solid brick:
Drill 18 mm x 90 mm deep rawplug
holes. Remove drilling template.
Clean rawplugs holes free from
dust. Inject FIS VS 150 C injection
mortar or similar from cartridge
(put on site). The manufacturer's
instructions must be followed. Insert
anchor sleeve.

Warning! Protect M 10 inner thread.
Observe curing time, screw firmly
to wall. Protect screws with anti-
corrosion grease.

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HEWI Heinrich Wilke GmbH